

# Alcavis Bleach-Wipe 1:10

## SURFACE DISINFECTANT

### A Consistent and Accurate Concentration of Bleach Every Time for:

- Medical, Dental & Lab Counters
- Exam Tables & Carts
- Point of Care Equipment
- Sinks & Counter Tops
- Treatment Rooms
- Lockers

#### Terminal Cleaning:

- Blood Spills
- Body Fluids



### EPA Registered Disinfectant\*

- Presaturated towelette for hard surface disinfection-3 organism (Staphylococcus aureus, Salmonella enterica, Pseudomonas aeruginosa).
- Tuberculocidal Test: AOAC Tuberculocidal Activity of Disinfectants.
- HIV: Viral efficacy of presaturated towelettes for hard surface disinfection (HIV-1 virucidal)



#### CONTACT US

Angelini Pharma Inc.

8322 Helgerman Court - Gaithersburg - MD 20877 -USA

1 (800) 726-2308

[www.Angelini-us.com](http://www.Angelini-us.com)

# Alcavis Bleach-Wipe 1:10

## SURFACE DISINFECTANT

### Protocol for use

- Always use personal protective equipment (gloves)
- Open wipe packet and remove pre-moistened towelette
- Wipe the surface to be disinfected completely, and allow surface to dry
- Discard used towelette

EPA-registered bactericidal and tuberculocidal wipe (#65787-2)

Meets CDC recommendations and OSHA Standard Precautions for bloodborne pathogen decontamination at 5,000 ppm NaOCl

Stable shelf life of 12 months



### Active Ingredient:

Sodium Hypochlorite: 0.525%

### Inert Ingredient:

99.4750%

Data on file\*

### Available in the following sizes:

1 - 8" x 10" towel package - 100/box - **Product code:** 09041-100

1 - 8" x 5" towel package - 140/box - **Product code:** 09053-140



### CONTACT US

Angelini Pharma Inc.

8322 Helgerman Court - Gaithersburg - MD 20877 -USA

1 (800) 726-2308

[www.Angelini-us.com](http://www.Angelini-us.com)

20-003-00 Rev 00 4/13