

# See-Luer™ CVC Cap

Designed to reduce the potential risk of contamination of central venous catheters in the hemodialysis population

- CDC “Scrub-the-Hub”\* compliant
- Addresses infection prevention
- Unique design
- Disposable
- Easy-to-use
- FDA 510K cleared



\*CDC Guidelines “Hemodialysis Central Venous Catheter Scrub-the-Hub Protocol (2012)” recommend removing the catheter cap and disinfecting the exposed threads and lumen as part of a two-step cleansing.

# See-Luer™ CVC Cap


## PROCEDURES FOR USE

### Initial See-Luer™ cap application and dialysis termination:

1. Open package and remove the two See-Luer™ Caps.
2. Using aseptic technique, remove the white cover and place See-Luer™ Cap on the end of the CVC. Push into place.
3. Twist collar clockwise to gently tighten the cap and cover the threads.
4. This position will be maintained when dialysis is not taking place.

### To access the catheter threads:

1. Carefully twist the See-Luer™ Cap counter clockwise until the CVC limb threads are exposed.
2. Disinfect the threads with an appropriate disinfectant, Alcavis 50 is recommended.
3. When ready to initiate dialysis and connect blood lines, remove the cap completely and connect. Limit the exposure time of this step.

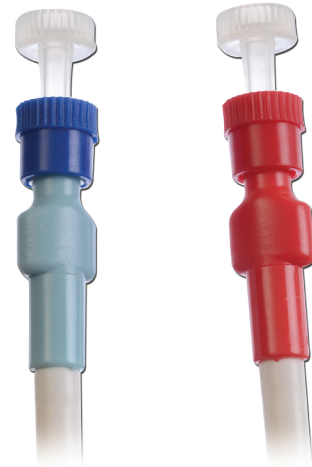

See-Luer™ Cap in CLOSED Position.



See-Luer™ Cap in OPEN Position.

**Available size:**

**200 sets/box - Product code: 501302**



#### CONTACT US

**Angelini Pharma Inc.**

8322 Helgerman Court - Gaithersburg - MD 20877 -USA

1 (800) 726-2308

[www.Angelini-us.com](http://www.Angelini-us.com)

10-006-00 rev 00 7/14