


Chronic wounds **can't** wait

BIOPAD™

PROVIDES THE POWER OF PURE COLLAGEN
FOR FAST HEALING

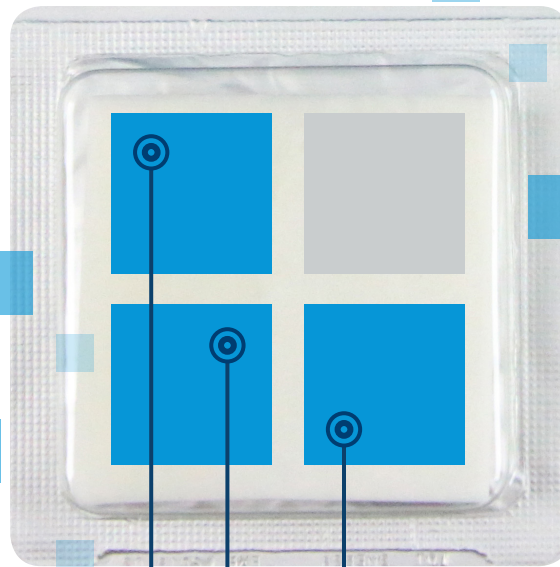

**THE OPTIMAL STRUCTURE FOR
HARD-TO-HEAL WOUNDS**

The dimension and thickness of the dressing perfectly sustain any cell movement during the healing process retaining the same overall native structure.

**100% PURE NATIVE EQUINE
TYPE I COLLAGEN**

No fillers, just pure collagen to stimulate proliferation of fibroblasts and the formation of granulation tissue.

**HIGHEST COLLAGEN CONTENT
ON THE MARKET**

With 5 times the standard amount of collagen, enhancing regeneration tissues, naturally increases the rate of healing reducing the frequency of dressing changes.



PRESENTATION AND SIZES

BioPad is packaged in a transparent waterproof blister pack and is sterilized by gamma irradiation.

2" x 2" - 3 per box | Order Code: 132622

4" x 4" - 1 per box | Order Code: 132644B

Biopad is covered by Medicare part B - HCPCS code: A6021

PREPARATION

Prepare the wound bed per the facilities wound care protocol and debride the wound when necessary.

Hydrate BioPad with sterile saline if the wound has limited exudate. (BioPad will turn into a gel consistency)

If the wound has heavy exudate, rinse out the wound bed with saline prior to application. (Do not hydrate BioPad prior to application)

APPLICATION

Cut BioPad, if necessary, to fit the size of the wound.

Apply directly to the wound bed, covering the entire wound; do not overlap the edges of the wound.

Apply a secondary dressing per physician protocol to cover the BioPad.

REAPPLICATION

Depending on the amount of exudate, BioPad can be reapplied every day.

If the wound has a small amount of exudate, BioPad can be reapplied every 48 hours.

BioPad will absorb into the wound bed over time.

INDICATIONS

BioPad is indicated for all chronic non-healing wounds that are free of necrotic tissue.

CONTRAINDICATIONS

Do not use on patients with a known hypersensitivity to collagen.



CONTACT US

Angelini Pharma Inc.

8322 Helgerman Court - Gaithersburg - MD 20877 - USA

1 (800) 726-2308

www.Angelini-us.com