

# ExSept Plus

## WOUND CLEANSER

### Indications for Use:

- Diabetic Foot Ulcers
- Venous Stasis Ulcers
- Pressure Ulcers

### Can be used in conjunction with:

- Sharp Debridement
- Negative Pressure Wound Therapy (NPWT)
- Preparation of wound site for skin equivalent or biologic dressing
- Irrigation of the wound site
- Compression dressings



### Protocol for Use:

1. Expose wound and place a chux pad under the area to absorb any extra solution.
2. Irrigate the wound bed with the ExSept Plus to remove any debris or exudate (for slightly higher pressure irrigation, a 35ml syringe with a 19 gauge needle will deliver approximately 8 PSI).
3. Open gauze pads and saturate with 8 – 12ml of ExSept Plus – gauze should be well saturated.
4. Place the gauze in the wound bed for 2 to 5 minutes; do not overlap onto the wound edges.
5. To remove the gauze, work in a circular motion toward the wound edge to remove any exudate.
6. Debride or dress the wound as indicated once the gauze has been removed.



#### CONTACT US

Angelini Pharma Inc.  
8322 Helgerman Court - Gaithersburg - MD 20877 -USA  
1 (800) 726-2308  
[www.Angelini-us.com](http://www.Angelini-us.com)