



Are clotted and blocked dialyzer headers affecting your reuse numbers?

- Designed to clean dialyzer headers without damaging dialyzer fibers.
- Reusable.
- Consistent with AAMI Standard "Reuse of Hemodialyzers".
- Improves the rinsing of dialyzer headers by spraying two lateral jets of RO water to loosen residual blood and other debris from the headers, header spaces, and header caps of multiple-use hemodialyzers.



A cost effective and compliant method of cleaning dialyzer headers without damaging dialyzer fibers.



The ASSIST® Header Cleaner is used to clean residual blood and other debris from the headers, header spaces, and header caps of multiple-use hemodialyzers prior to an approved reprocessing process for hemodialyzers.

WARNING:

Use full Personal Protective Equipment, including durable gloves and protective clothing, when handling dialyzers and using the Assist device as recommended in the AAMI Standard "Reuse of Hemodialyzers" RD47; 2002. Personnel shall wear eye protection when performing steps that may result in spill or splashes.

CAUTION:

- Ensure correct Assist device is compatible with dialyzer used.
- If any leaks are noted at the header port or the Quick Connect, tighten the connections, or discard the Assist and use a new Assist.
- Use only under the direction of a Physician.



Available in the following size:

50 units/bag - Product Code: 50910

For use with Gambro Polyflux® & Asahi® Series dialyzers

50 units/bag - Product Code: 50911

For use with Bellco®, Baxter® & Fresenius® Series dialyzers.



CONTACT US

Angelini Pharma Inc.

8322 Helgerman Court - Gaithersburg - MD 20877 -USA

1 (800) 726-2308

www.Angelini-us.com