


# Alcavis Bleach-Wipe 1:100

## DILUTED BLEACH WIPE FOR SURFACE CLEANING

Ideal for cleaning hard non-porous surfaces including the following:

- Medical, Dental & Lab Counters
- Exam Tables & Carts
- Dialysis Machines
- Point of Care Equipment
- Sinks and Counter Tops
- Treatment Rooms
- Lockers

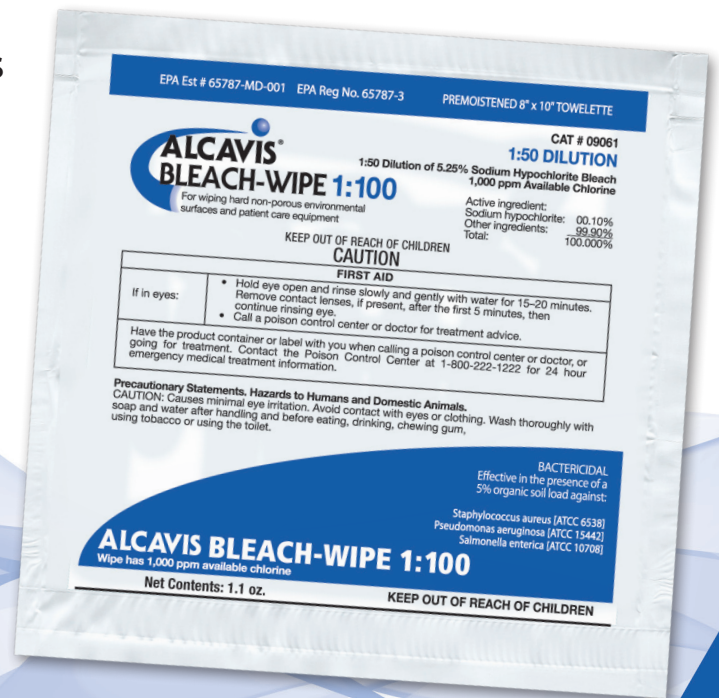

**Bleach-Wipes are available in single-towel or double-towel packages.**

The double-towel packages are convenient for wiping down two different surfaces, large surfaces and re-wiping surfaces that may be dirty.



ANGELINI

### CONTACT US

Angelini Pharma Inc.

8322 Helgerman Court - Gaithersburg - MD 20877 -USA

1 (800) 726-2308

[www.Angelini-us.com](http://www.Angelini-us.com)

# Alcavis Bleach-Wipe 1:100

## DILUTED BLEACH WIPE FOR SURFACE CLEANING

### PROTOCOL FOR USE

- Open wipe packet and remove pre-moistened towelette
- Wipe the surface completely to be cleaned and allow surface to air dry and discard used towelette
- This product is an EPA-registered disinfectant

Stable shelf life of 18 months



### Active Ingredient:

Sodium Hypochlorite: 0.0525%

### Inert Ingredient:

99.9475%

### Available in the following sizes:

1 towel package - 100/box - **Product code:** 09031-100

2 towel package - 50/box - **Product code:** 09032-50



### CONTACT US

Angelini Pharma Inc.

8322 Helgerman Court - Gaithersburg - MD 20877 -USA

1 (800) 726-2308

[www.Angelini-us.com](http://www.Angelini-us.com)

20-005-00 rev 01 1/16