


Dermacoll-DA™

WOUND MANAGEMENT

Dermacoll-DA has a 3-layer construction composed of an absorbent hydrocolloid coated layer backed by a thick, soft, smooth elastic and breathable layer of polyurethane.

This 3-layer combination enhances the total fluid handling capacity of the dressing while ensuring moisture is regulated within the wound.

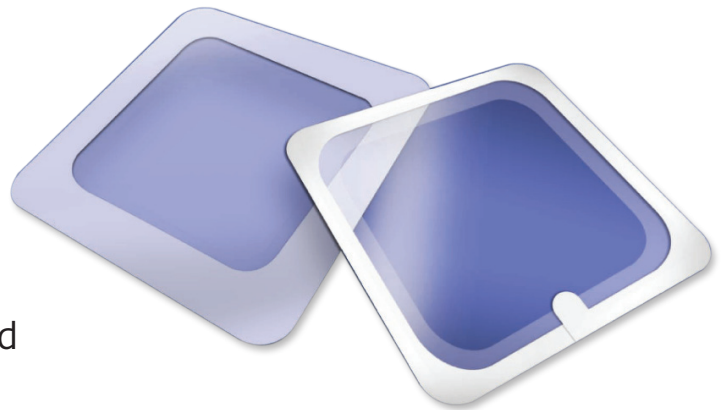

INDICATIONS

Dermacoll-DA is indicated for wound management of slight to moderate exudate, wounds that are granulating, or have areas of necrosis.

- Chronic Wounds
- Pressure Ulcers
- Diabetic Ulcers
- Skin Donor Sites
- Post-Operative Surgical Wounds
- Abrasions

Dermacoll-DA is unique in its ability to absorb fluid and transport it to the polyurethane foam matrix and the breathable polyurethane film outer layer.



ANGELINI

CONTACT US

Angelini Pharma Inc.

8322 Helgerman Court - Gaithersburg - MD 20877 -USA

1 (800) 726-2308

www.Angelini-us.com

Dermacoll-DA™

WOUND MANAGEMENT

APPLICATION

- Gently cleanse the wound with ExSept Plus or SilverStream (per guidelines). Dry the surrounding skin
- Debride if necessary
- Select the correct size, allowing a minimum of 1 ¼ (3.2cm) margin beyond the reddened area
- Remove the release paper from the dressing, minimizing contact with the adhesive surface
- Place the dressing directly over the wound and apply with a rolling motion; do not stretch
- Press and smooth the dressing edges to ensure adherence and a firm seal



Available in the following sizes:

Dermacoll-DA – 4" x 4" 10/box – **Product Code:** 317744

Dermacoll-DA – 8" x 8" 3/box – **Product Code:** 317788



CONTACT US

Angelini Pharma Inc.

8322 Helgerman Court - Gaithersburg - MD 20877 -USA

1 (800) 726-2308

www.Angelini-us.com